

On 26/05/2026.

To the attention of:
The Pharmacist
The materiovigilance correspondent
Of the ANSM

URGENT – SAFETY NOTICE FSN 202605-01

3-way Stopcock REF : PER3Y LOT 2501BH400

Possible patient incident related to the 3-way Stopcock contained in the set ABSOSOINS PERFUSION - Voluntary recall of medical devices

Dear Sir/Madam,

We have identified a problem with a batch of 3-way Stopcocks REF : PER3Y contained in the ABSOSOINS PERFUSION infusion set of our client Laboratoire Marque Verte.



Compliant injection site

A weakness has been detected at the Y injection site composed of two parts, one made of plastic, the other of isoprene rubber. Cracking has been observed, which may cause breakage of the upper part of the injection site and detachment of the isoprene rubber part.



In the case of a clear break of the upper part of the Y injection site, the risk is limited for the patient as the defect is easily detectable by nursing staff before use.

In the case of a minor crack that is difficult to detect, the crack may expand during drug injection, potentially causing a risk of contamination or air entry into the infusion line, with a serious risk to the patient's health.

Injection site with breakage

The concerned stopcocks are exclusively contained in the ABSOSOINS PERFUSION sets of our client.



Please find below the LOT OF THE INFUSION SET containing the stopcock concerned by the recall as well as the LOT OF THE STOPCOCK.

REFERENCE	EAN	LOT	Remark
SETPERF-VV ABSOSOINS PERFUSION SET FOR FRAGILE OR SUBCUTANEOUS VEINS	3615370005509	2506Z200	Set containing the PER3Y stopcock, batch 2501BH400

We ask you to immediately stop using these devices.

We kindly ask you to check your stock status in order to determine if :

1. You are still in possession of one or more impacted products
2. If so, stop the sale and/or its use



BQ Plus Medical Co., Ltd.

No. 18, Che Ye Road, Che Dun Town, Songjiang, 201611 Shanghai, P.R. China

Tel: 0086-21-57609106 [http:// www.bq-md.com](http://www.bq-md.com)

At our request, Laboratoire Marque Verte will contact each of its customers by email and will send them a safety notice and a return form

We regret this incident and guarantee our full commitment to supporting you.

We thank you for ensuring the proper dissemination and application of this safety notice to your patients

The ANSM has been duly informed of this communication.

Remaining at your disposal, please accept, Dear Sir/Madam, the expression of our most sincere greetings.

BQ Plus Medical Co., Ltd.

Name: Bonnie Xu

Position: QA&Regulation Manager

Date: 26/05/2026

Email: mr@bq-medical.com

**BQ Plus Medical Co., Ltd.**

No. 18, Che Ye Road, Che Dun Town, Songjiang, 201611 Shanghai, P.R. China

Tel: 0086-21-57609106 [http:// www.bq-md.com](http://www.bq-md.com)

Ref.: F001155A

A Field Safety Notice Customer Reply Form

1. Field Safety Notice (FSN) information	
FSN Reference number*	202605-01
FSN Date*	2026.05.26
Product/ Device name*	INFUSION SET FOR GRAVITY FEED
Product Code(s)	PER3Y
Batch/Serial Number (s)	2501BH400

2. Customer Details	
Account Number	
Healthcare Organisation Name*	
Organisation Address*	
Department/Unit	
Shipping address if different to above	
Contact Name*	
Title or Function	
Telephone number*	
Email*	

3. Customer action undertaken on behalf of Healthcare Organisation				
☐	I confirm receipt of the Field Safety Notice and that I read and understood its content.	Customer to complete or enter N/A		
☐	I performed all actions requested by the FSN.	Customer to complete or enter N/A		
☐	The information and required actions have been brought to the attention of all relevant users and executed.	Customer to complete or enter N/A		
☐	I have returned affected devices - enter number	Qty:	Lot/Serial Number:	Date Returned (DD/MM/YY):
		Qty:	Lot/Serial Number:	Date Returned(DD/MM/YY):

	of devices returned and date complete.	N/A	Comments:
☐	I have destroyed affected devices – enter number destroyed and date complete.	Qty:	Lot/Serial Number:
		Qty	Lot/Serial Number:
		N/A	Comments:
☐	No affected devices are available for return/ destruction	Customer to complete or enter N/A	
☐	Other Action (Define):		
☐	I do not have any affected devices.	Customer to complete or enter N/A	
☐	I have a query please contact me (e.g. need for replacement of the product).	Customer to enter contact details if different from above and brief description of query	
Print Name*		Customer print name here	
Signature*		Customer sign here	
Date*			

4. Return acknowledgement to sender	
Email	mr@bq-medical.com info@bq-medical.com
Customer Helpline	+86 21 57609106-816
Postal Address	No. 18, Che Ye Road, Che Dun Town, Songjiang, 201611 Shanghai, P.R. China
Web Portal	https://www.bqplusmedical.com/
Fax	/
Deadline for returning the customer reply form*	2026.06.20

Mandatory fields are marked with *



BQ Plus Medical Co., Ltd.

No. 18, Che Ye Road, Che Dun Town, Songjiang, 201611 Shanghai, P.R. China

Tel: 0086-21-57609106 [http:// www.bq-md.com](http://www.bq-md.com)

It is important that your organisation takes the actions detailed in the FSN and confirms that you have received the FSN.

Your organisation's reply is the evidence we need to monitor the progress of the corrective actions.