



Dexcom, Inc. | Corporate Headquarters  
6340 Sequence Drive | San Diego, CA 92121  
888.738.3646 | dexcom.com

## **Urgent Warning - Dexcom G7 iOS App version 2.7 – Mandatory Upgrade**

**US-MF-000010694 – UDI: 00386270001863**

### **Description of the Problem:**

A bug was identified within app version 2.7 that can cause the app to enter a state where it stops updating the estimated glucose values, which could result in alerts not triggering. You're using an older version of the Dexcom G7 iOS app, which may not function as intended.

### **Risk to Health:**

If a user is unaware that glucose values provided by the app are not updating and that glucose alerts will not trigger, there is potential for the missed detection of a hyperglycemic or hypoglycemic event, or a treatment decision made based on incorrect data. Through February 2025, 111 complaints have been received, associated to this issue. Of those, there have been no serious injuries.

### **Recommended Customer Action:**

Upgrading from app version 2.7 is mandatory and you will not be able to use app version 2.7 after May 19, 2025. Update your app now to continue using the app:

1. Tap **Update App** to go to the app store
2. Install the latest Dexcom G7 iOS app version
3. Open the Dexcom G7 iOS app

If you need assistance, contact Technical Support. Local Tech Support contact information is available at [Dexcom.com](https://dexcom.com)

Adverse reactions or quality problems experienced with the use of these products should be reported to the National Competent Authority in your area.