PAUL HARTMANN AG
Paul-Hartmann-Strasse 12
89522 Heidenheim

P.O. Box 1420
89504 Heidenheim
Germany

Phone: +49 (0) 7321 36-0
Fax: +49 (0) 7321 36-3636
www.hartmann.info

PAUL HARTMANN AG, P.O. Box 1420, 89504 Heidenheim, Germany

Mr Sample
Company
27 Colmore Row
Birmingham
England
B3 2EW

Your ref.	Contact
Your ref.	Mr Sample
Your letter of	Phone
Date	+49 7321 36
Our Ref.	Fax
Our Ref.	+49 7321 36
Date	E-Mail
Date	john.sample@hartmann.info

URGENT: FIELD SAFETY NOTICE - Medical Device Safety Advisory Notice

Kocher forceps, included in the HARTMANN gynaecology and obstetrics sets

Dear Medical device safety officers,

PAUL HARTMANN AG stands for safety and reliability of its products and also feels responsible for information related to the use of these products.

With this letter, we would like to inform you that PAUL HARTMANN AG has initiated a field safety corrective action for the HARTMANN Gynecology and Obstetrics sets with Kocher forceps listed in the table in Appendix 1.

REASON FOR THE Field Safety Correction:

After receiving complaints and due to a potential serious incident, PAUL HARTMANN AG has initiated a voluntary field safety corrective action as a matter of precaution.

This action is based on potential serious incidents while using forceps for clamping too large objects compared to the size of the clamp. Peha-instrument Surgical Hemostatic forceps and HARTMANN Surgical Instruments Surgical Hemostatic forceps are intended to be used as surgically invasive, single-use, surgical steel instruments for short-term treatment in the context of medical interventions such as holding and removal of foreign objects, holding and clamp of tissue and tubes, occlusion of vessels, manipulation and removal of tissue on humans used by healthcare professionals. Clamping of umbilical cords may make clamping difficult, and carries the risk that the forceps may come loose or break due to the physical nature of the umbilical cord.

POTENTIAL HAZARDS:

The use of Kocher forceps, included in the HARTMANN gynecology and obstetrics sets listed in the table in Appendix 1, for cord clamping may make clamping difficult, and carries the risk that the forceps may come loose or break due to the physical nature of the umbilical cord.

In fact, such use could result in significant blood loss in the infant, which could have serious consequences.

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Sitz Heidenheim
Amtsgericht Ulm HRB 661090
Registered Office Heidenheim
Commercial Register of the District Court of Ulm file no. HRB
661090

PAUL HARTMANN AG
Paul-Hartmann-Strasse 12
89522 Heidenheim
P.O. Box 1420
89504 Heidenheim
Germany

Phone: +49 (0) 7321 36-0
Fax: +49 (0) 7321 36-3636
www.hartmann.info

The HARTMANN OB/GYN procedure trays affected by this field safety notice are no longer being produced with this type of Kocher forceps. The replacement clamps reduce the incidence of breakage and inadvertent opening of the Kocher clamp when used properly with blood vessels.

In addition, we are revising our instructions for use for the affected HARTMANN Kocher forceps, which specify the intended use that excludes clamping of umbilical cords.

Further, the sets that are in the market need to be labelled with respective information that excludes clamping of umbilical cords. These instructions are enclosed as Annex 3 of this letter

REQUIRED ACTION:

Step 1: Please take note of this safety information and inform all users in your facility.

Step 2: Please check your inventory physically as a matter of urgency and immediately quarantine the affected HARTMANN gynecology and obstetrics sets listed in the table in Annex 1.

Step 3: Please complete the acknowledgement of receipt (Annex 3) and return it by email as soon as possible, but not later than 03 March 2025. Please indicate the quantity of HARTMANN gynecology and obstetrics sets affected in your inventory so that PAUL HARTMANN AG can prepare the required quantity of 'warning labels'. With this acknowledgement, you also confirm that you have received the report and that you have understood and will implement the letter.

Step 4: Your HARTMANN sales representative will relabel the affected products in your inventory. Appointments will be arranged with you shortly to label the boxes of the affected sets.

Step 5: If you no longer have any of the affected products in stock, please complete the acknowledgement of receipt (Annex 3) and return it by email as soon as possible, but no later than 03. March 2025.

With this acknowledgement of receipt, you also confirm that you have received the notification and that you have understood and will implement the letter.

Thank you for your cooperation. We apologise for the additional work that this FSMA has caused you.

Kind regards

PAUL HARTMANN AG

Site or department name (if necessary)

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P.O. Box 1420
89504 Heidenheim
Germany

Phone: +49 (0) 7321 36-0
Fax: +49 (0) 7321 36-3636
www.hartmann.info

Safety notice
Do not use Kocher forceps for umbilical cord clamping respectively only under supervision.

Annex 3: – Customer Response Form:

[Enter Customer address]

Urgent Field Safety Notice – Kocher forceps, included in the HARTMANN gynaecology and obstetrics sets

We hereby confirm receipt of the Field Safety Notice and the forwarding to all those who need to be aware within our organization or to any organization where the potentially affected products have been transferred.

The result of an examination of our stocks is as follows:

Please indicate:

☐ after thorough examination there is **no stock** of the affected products

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Registered Office Heidenheim

Commercial Register of the District Court of Ulm file no. HRB
661090

PAUL HARTMANN AG
Paul-Hartmann-Strasse 12
89522 Heidenheim

P.O. Box 1420
89504 Heidenheim
Germany

Phone: +49 (0) 7321 36-0
Fax: +49 (0) 7321 36-3636
www.hartmann.info

Helps. Cares. Protects.

☐ **existing Sets on stock must be labeled** as per HARTMANN instructions (fill table below in case, additional products/ labels needed)

REF (article number)	trade name	LOTs	stock quantity	Desired Action for product
„number“	„product name“ (customer individual)	„number “	„Number of primary packs “	“Return / Label / Replacement needed”

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