ALBANIAN	NJOFTIM SIGURIE PAJISJA MBROJTËSE
ARABIC	إشعار السلامة طقم إصلاح
BRAZILIAN	Aviso de segurança Kit tampão
BULGARIAN	Предупреждение за безопасност Буферен набор
CHINESE	安全通知 缓冲套件
CZECH	Bezpečnostní oznámení Sada tlumiče
DUTCH	Veiligheidswaarschuwing Bufferkit
ENGLISH	FIELD SAFETY NOTICE BUMPER KIT
FRENCH	Avis de sécurité Kit tampon
GERMAN	Sicherheitshinweis Pufferset
HUNGARIAN	Biztonsági felhívás Ütköző készlet
ITALIAN	Avviso di sicurezza Kit tampone
LITHUANIAN	Saugos įspėjimas Stūmoklio rinkinys
NORWEGIAN	Sikkerhetsvarsel Bufferset
POLISH	Zalecenia bezpieczeństwa Zestaw buforowy
PORTUGUESE	Aviso de segurança Kit tampão

General Medical Merate S.p.A.

Stabilimento

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Via Partigiani, 25 - 24068 SERIATE - BG - ITALIA - Tel. +39 035 4525311 - Fax +39 035 299537/297787

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Cert. N.
9124.GNM2

FIELD SAFETY NOTICE BUMPER KIT

Dear Customer,
the purpose of this letter is to inform you that GMM has decided to intervene on its products in order to guarantee their reliability over time.

Products involved: system from serial number 00811 to serial number 01672 included, according to the following names:

<i>Name</i>	<i>Brand</i>
Clisis Systems	Mecall
EIDOS RF439	Mecall
Discovery RF180	General Electric

Description:

A transmission screw for the elevation and tilting movements of the patient tabletop could show a reduction in its reliability over time. This could be determined by a series of multiple causes including, by way of example, a possible collision of the table in the tilting phase with external objects, improper use of the device, possible mechanical misalignments and/or by a particular and extraordinarily intense use of the equipment.

Corrective action:

Code 92251 - bumper kit – has been created for the purpose of being implemented on the equipment installed base, in accordance to the serial numbers provided here above.
The kit contains all the parts and instructions necessary to replace the components.

Warning!

The instructions must be filled in completely and returned to the manufacturer by e-mail or fax to the following address:

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Cert. N.
9124.GNM2

or fax to **+39 035 29 77 87**

We would like to point out that, to the present date, no event has been reported of damage to property or major injuries to persons related to what is described in this communication.

Nevertheless, the actions described must be undertaken to mitigate a potential risk that could reduce the safe use of the equipment.

The kit will be progressively available and shipped within April 2024 and must be implemented progressively during the first ordinary/extraordinary maintenance and, in any case, **no later than 7 months** from receipt of the kit itself, prioritizing the devices according to the attached list:

<i>Priority</i>	<i>Device number</i>
1	Serial_Number_List_EN.pdf
2	Serial_Number_List_EN.pdf
3	Serial_Number_List_EN.pdf

As already indicated in the User Manual, in the Service Manual and in the instructions provided to the end user, please remember that any operation under the table must be avoided.

Counting on your attention in relation to this notice, GMM declines all responsibilities in case of non-application of the above-described measures within the period of time stated in this communication.

If the system has been sold and/or scrapped and it is therefore no longer in your possession, we invite you to forward this safety notice to the new owner, and tell us its name, or provide us with notification of the scrapping.

GMM technical assistance service is available for any further request for information and / or support (international_service@gmmspa.com)

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